



INSTRUCTIONS


TEST KIT CONTENTS:

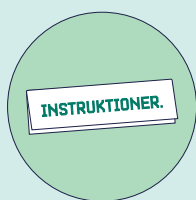


INSTRUCTIONS

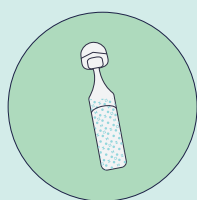


SALINE SOLUTION
AMPOULE

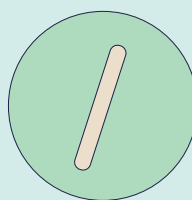


TONGUE SPATULA

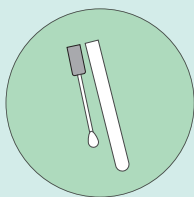

SWAB

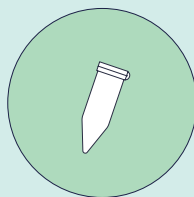

SALIVA
COLLECTION TUBE

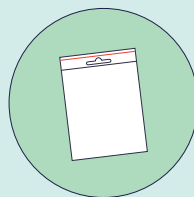

ZIP BAG



FORM FOR
SELF-REGISTRATION



Self-swab Kit COVID-19

Hello! This is Dynamic Code's COVID-19 self-swab kit. The swab takes only a few minutes to complete and will be sent to our lab for analysis. You will get the test results on 1177.se. See what applies to your region on 1177.se.

→ **Once you have taken the test, it must be delivered on the same day.**

Read through the full instructions before starting the test so you don't miss anything important.


PREPARATION



1. READ THE INSTRUCTIONS

Start by washing and drying your hands thoroughly or cleaning them with hand sanitizer.

Read through the full instructions before starting the test so you don't miss anything important.

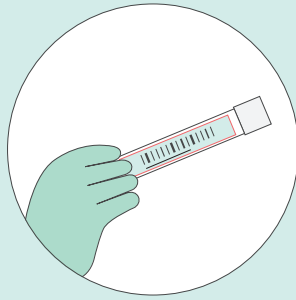

2. CHECK THE DATA

Remove the test tube with the swab and check for a bar code label.

If a label is missing please contact the place where your test kit was collected.




3. RINSE YOUR NOSE WITH SALINE SOLUTION

a) Blow your nose if you have mucus in the nose.

b) Twist off the seal at the top of the saline solution ampoule. Tilt your head back and drip about half of the solution into one nostril.

Drip the rest into the other nostril. Leave for a short while (10-15 seconds) and let the solution pass the throat by swallowing.

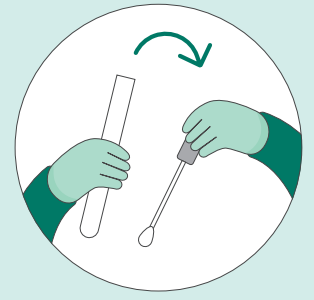

4. PREPARE FOR THROAT SWAB

a) Take the tube with the swab.

b) Make sure you have a mirror or mobile phone with a camera to look down your throat.

c) Unscrew the lid on the tube and remember not to touch the soft part of the swab.

SWABBING




5. SWAB THE TONSILS AND BACK OF THE THROAT

a) Look in the mirror. Tilt your head slightly back and open your mouth wide to get a clear picture of the tonsils.

b) Use the spatula to hold down the back of your tongue.

c) Rub the swab on the tonsils on both sides of the throat for 5-7 seconds. Try to avoid touching other parts of the mouth.

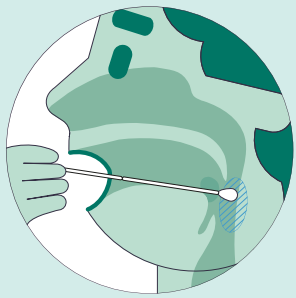

d) Now move the swab past the tonsils to the back of the throat. Scrape and rub the swab against the mucous membrane for 5-7 seconds. If you feel a vomit reflex, it means you're in the right place.

e) Proceed to step 6 without putting the swab down.




6. NASAL SWAB

a) Insert the soft part of the swab 2 to 3 centimetres into one nostril and rub it around the nostril for 5-7 seconds.

b) Swab in the same way in the other nostril.

c) Proceed to step 7 without putting the swab down.

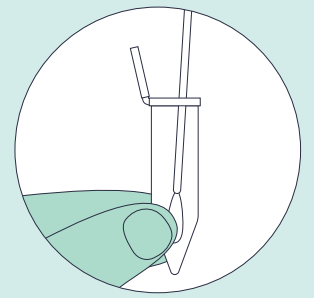

7. DIP IN SALIVA

a) Take the saliva collection tube and open the lid.


b) Spit 2-3 times into the saliva collection tube.

c) Dip the swab into the saliva and rotate a couple of turns.

d) Discard the saliva collection tube.


e) Proceed to step 8 without putting the swab down.

DELIVERY

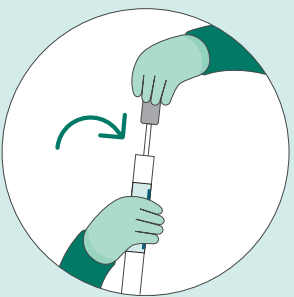

8. PUT THE LID ON


Put the swab back directly into the tube and press the lid firmly on.

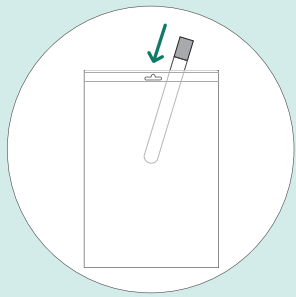

9. PACK THE SWAB

Place the test tube with the sampling swab in the zip bag and seal.

Store the bag with its contents in the fridge until you deliver the swab. You must do so on the same day.

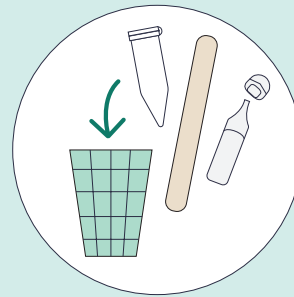

10. DISCARD OTHER MATERIALS

Toss the saliva collection tube, tongue spatula and saline solution ampoule into your regular household waste.

These should not be delivered with the swab.



11. DELIVER AND WAIT FOR RESULTS

Do not forget to register your test at 1177.se before submitting the sample.

See separate instruction for self-registering of sample.

Deliver the sample on the same day to the place where it was collected. You'll get the test results on 1177.se within three days. If you have not received a reply after three days, please order a new test and report the missing test result to support at 1177 e-services.